

HearID Quick Start — Installation Notes

These Quick Start Notes complement, but do not take the place of the manuals. We recommend reading the full installation manual before installing HearID. We also recommend reading the General User's manual and the manuals specific to each measurement module before testing any patients.

Where do I find the manuals?

- Before installation, the manuals (in pdf format) – including the installation manual – may be read off the installation CD. Insert the CD-ROM into the CD-ROM drive. If the installer starts automatically, cancel it. Go to **My Computer** and navigate to the **Manuals** folder on the CD.
- After installation, assuming the default location was used, the manuals are located in C:\Program Files\HearID\Documents. Use a PDF reader like Adobe Reader™ to load the manuals.

Computer Hardware and System requirements

- IBM™ compatible laptop computer that is EN60950 (IEC950) compliant.
- Microsoft Windows™ 2000, XP, or Vista.
- An available **PC-Card cardbus (Type II)** slot.
- Pentium™ class processor at least 1.30 GHz.
- At least 512 MB RAM.
- At least 10 GB hard drive.
- Color laptop screen and a CD-ROM drive.
- Recent version of Adobe Reader™ (for displaying the electronic manuals).

Installing HearID Software

- *Do not insert the PC-Card into the computer until the driver software installation is complete.*
- Start the installation process from a freshly re-booted computer with no other programs running.
- Insert the Mimosa Acoustics HearID CD-ROM into the CD-ROM drive.
- The setup program automatically runs. If it doesn't, manually start the installer by activating the Windows **Start** button, selecting **Run** and typing in **DriveLetter:setup.exe** (where DriveLetter is the letter for your CD-ROM drive, e.g., D:setup.exe) in the **Open** field.
- Follow the onscreen instructions to install the software.

Installing the PC-Card

- Install the Mimosa HearID software **before** inserting the PC-card.
- Insert the PC-card into a free cardbus slot in the computer.
- If you are using Windows XP, the **Add New Hardware** Wizard appears. The drivers for your PC-card are already installed. **Do not** connect to Windows Update to search for them.
- Follow the onscreen instructions. Choose **Continue Anyway** to the message that the driver has not passed Window's XP logo testing.

Software Activation

- Run HearID by double-clicking the **HearID icon** on the desktop or choose it from the **Start** menu.
- The first time the software runs, the **Activation** screen appears. There are two pieces of information necessary to properly activate your system—**Institution Name** and **Activation Key**. You will find both items on the CD case or the information will be provided by your vendor. Please enter them into the Activation screen exactly as they appear on the label.